



# Exocad libraries installation procedure


1: Visit our website and click on: downloads and after on CAD library.

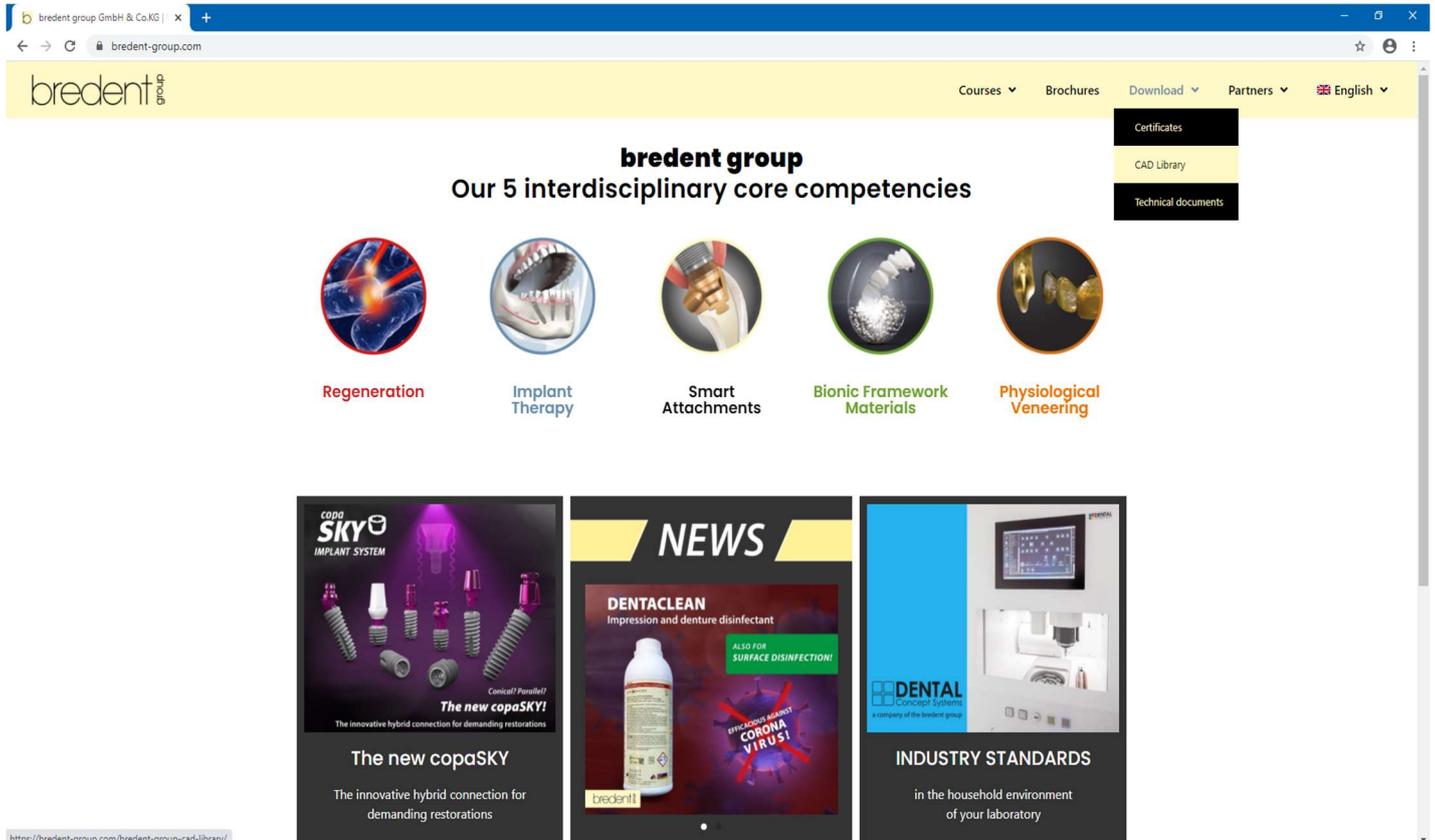


2: Accept our terms to continue.

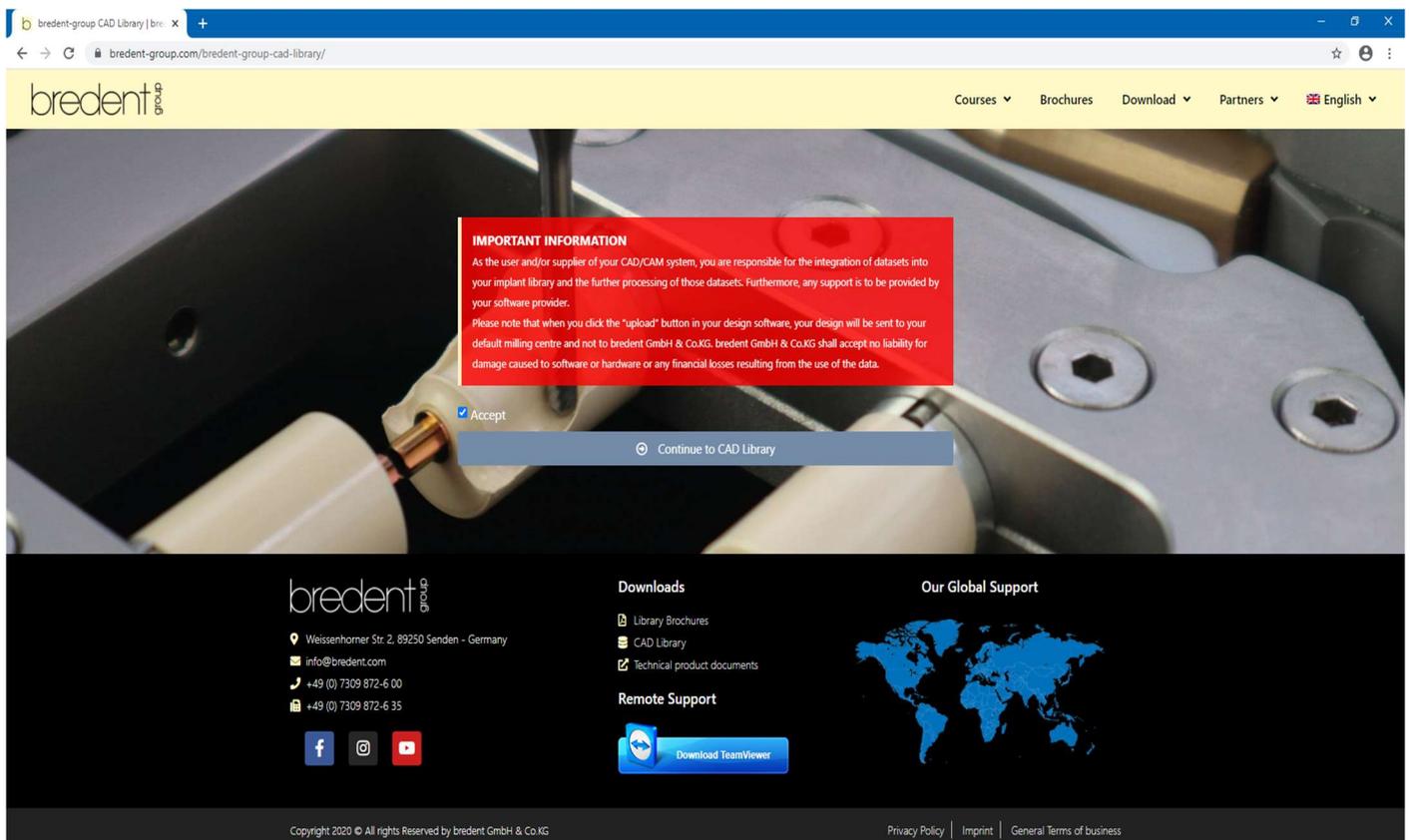


3: Choose the files to download. The download will start immediately.

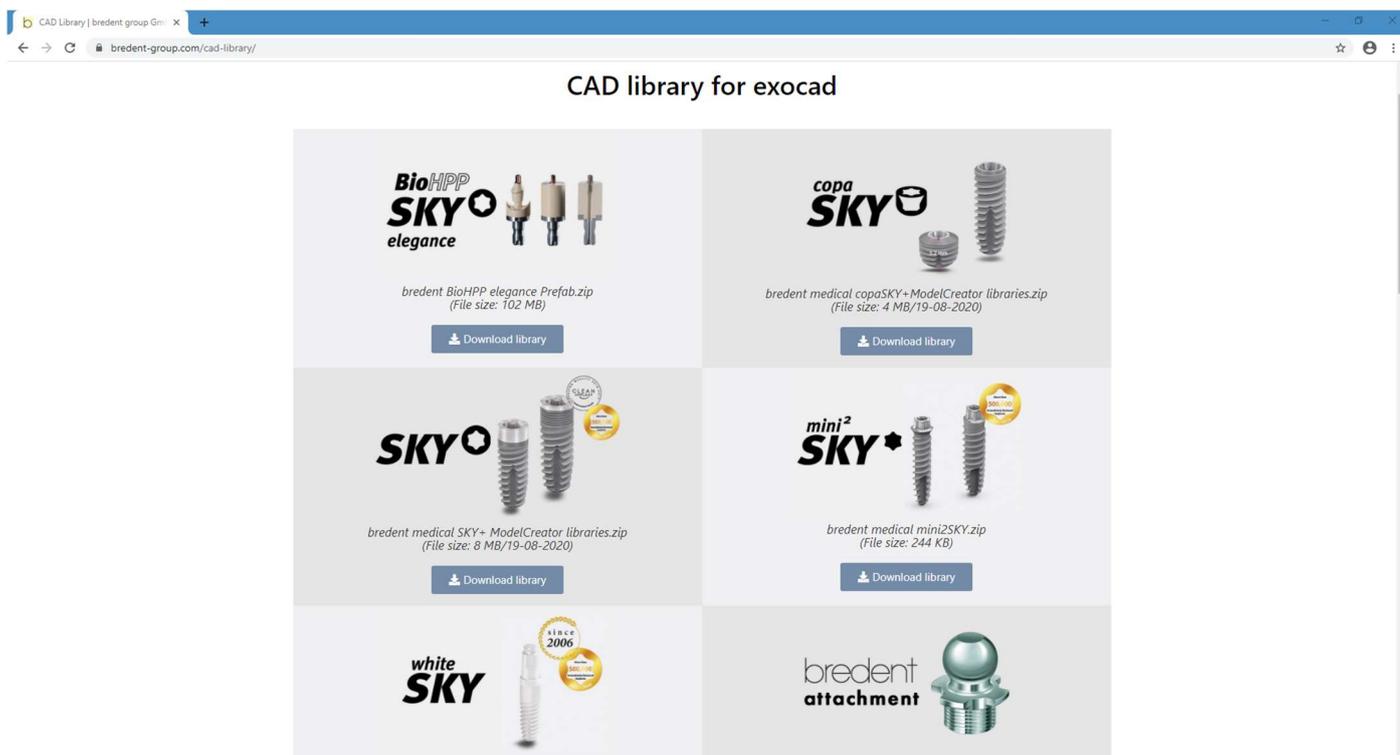


4: Unzip all the files (this is just an example with implant libraries but with model creator libraries, implant libraries, attachment libraries and bar libraries the procedure is the same).

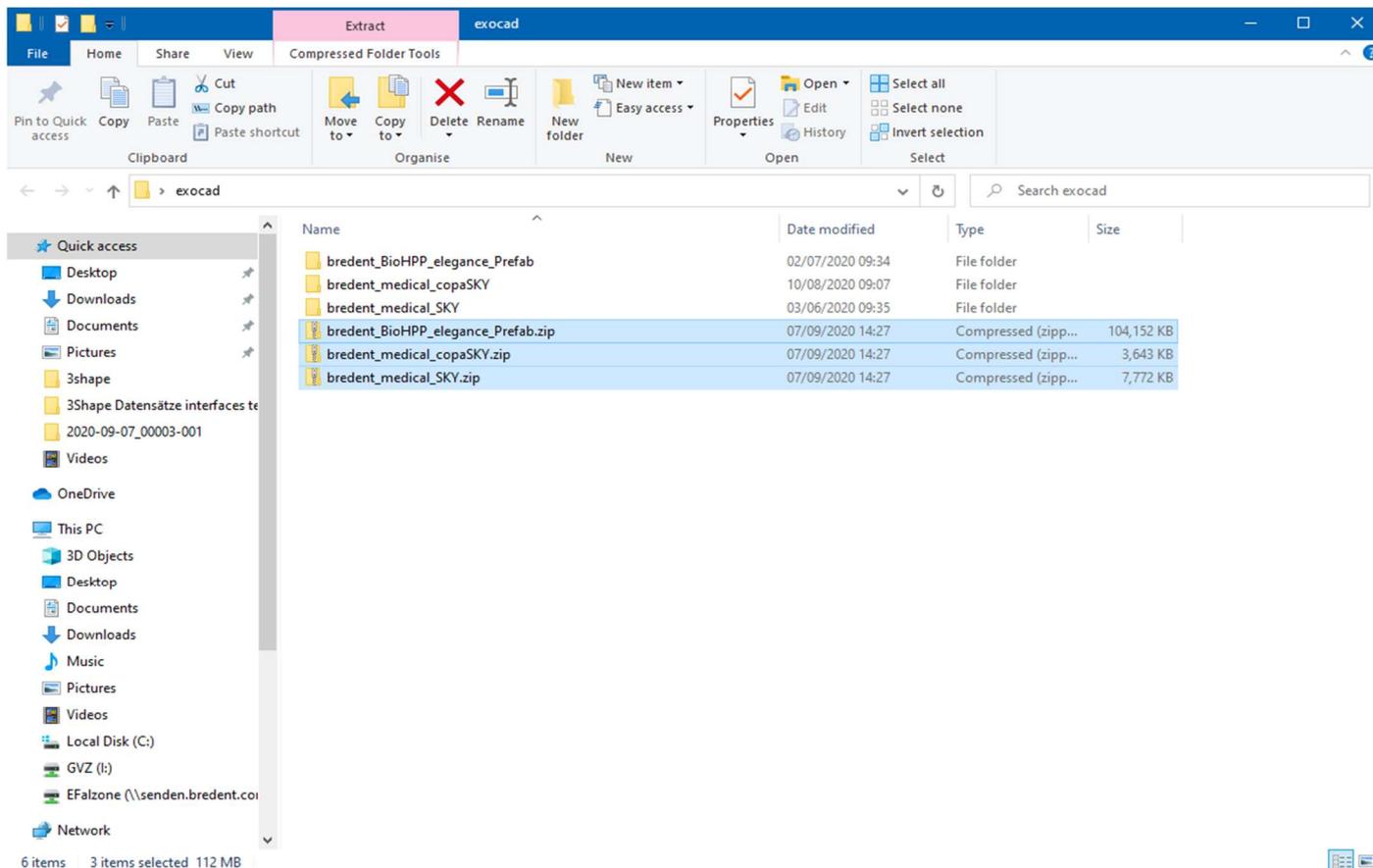


5: Select and copy the extracted files.

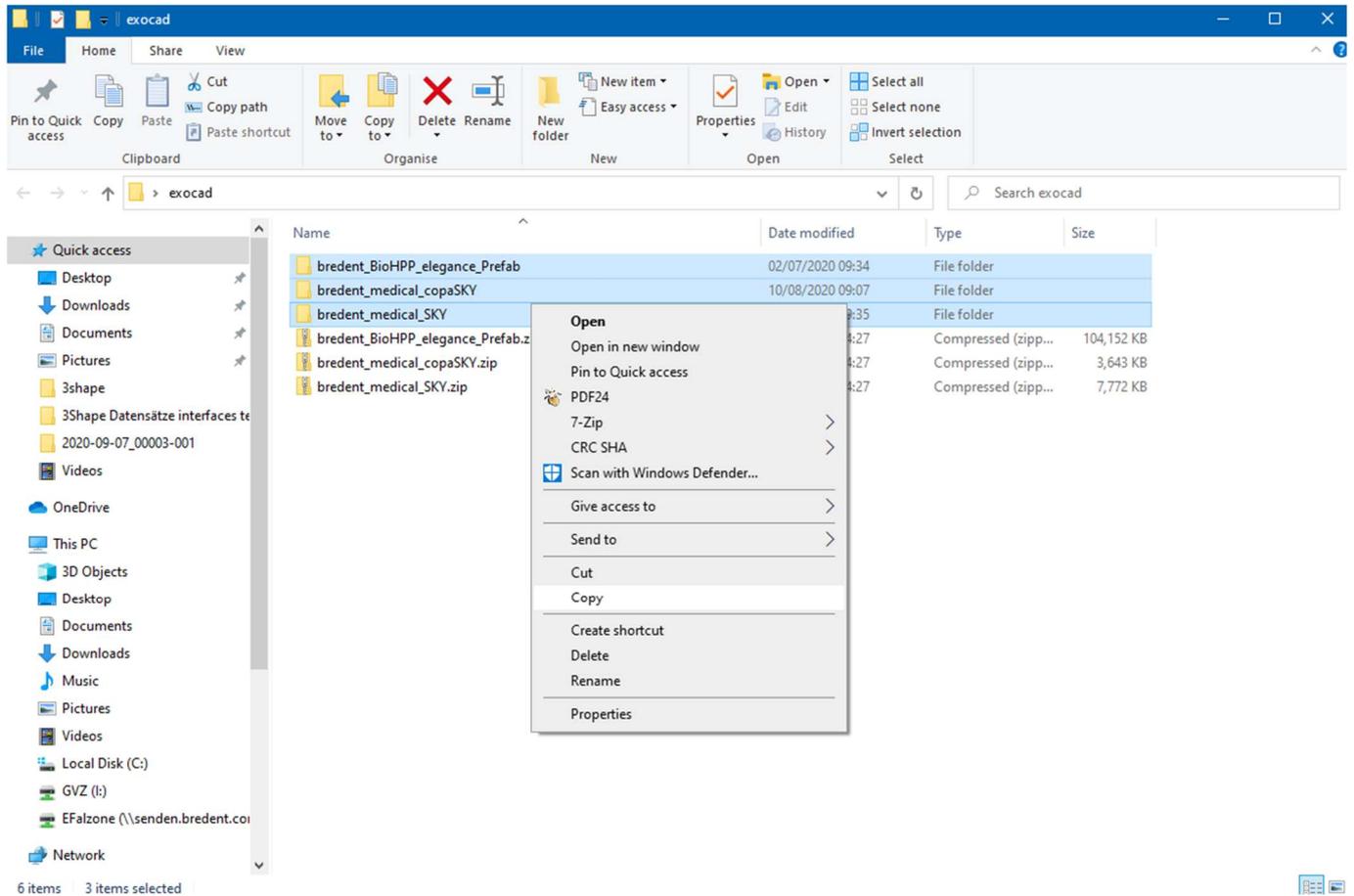


6: Open the Exocad installation folder on your computer.

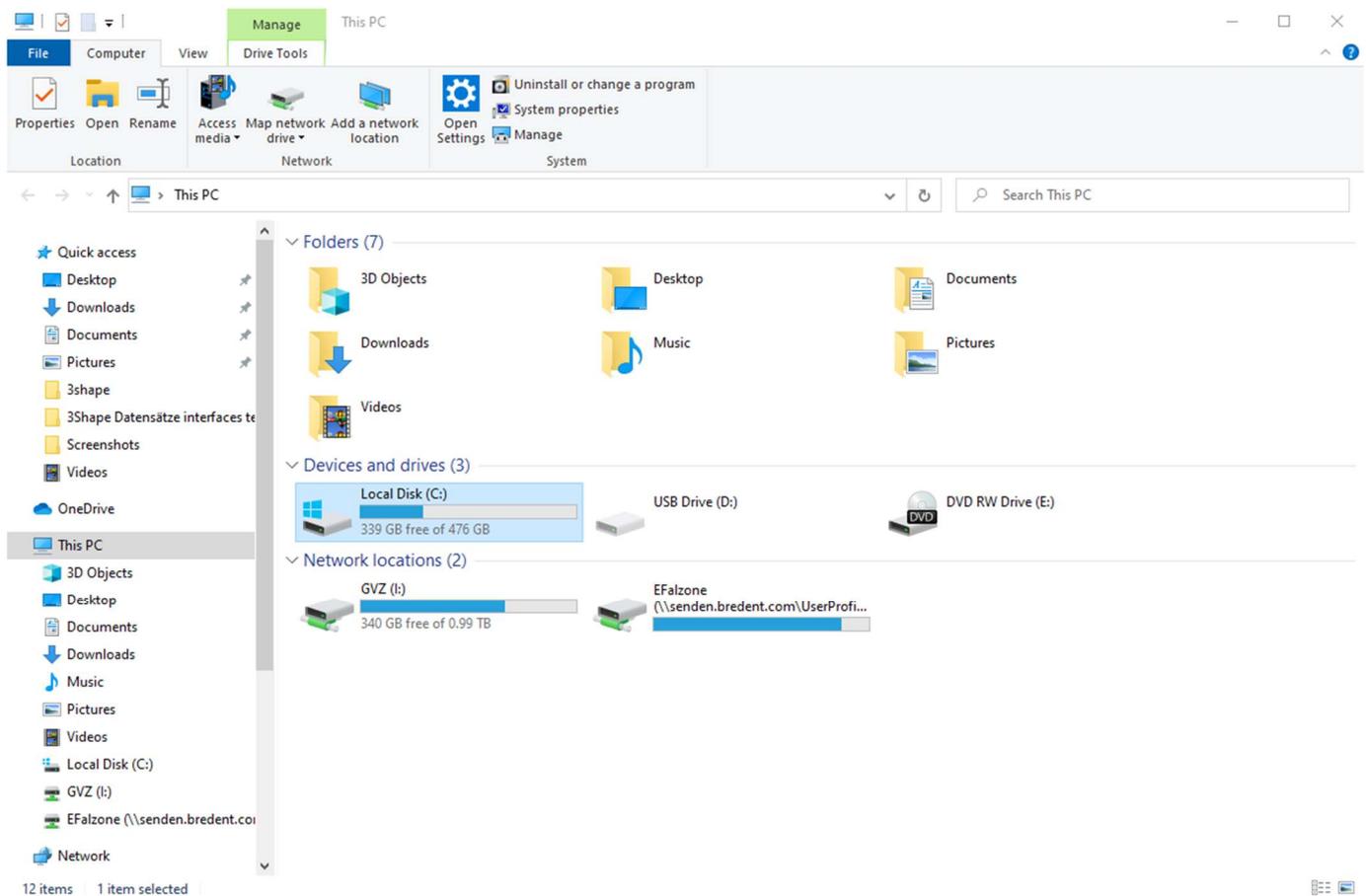

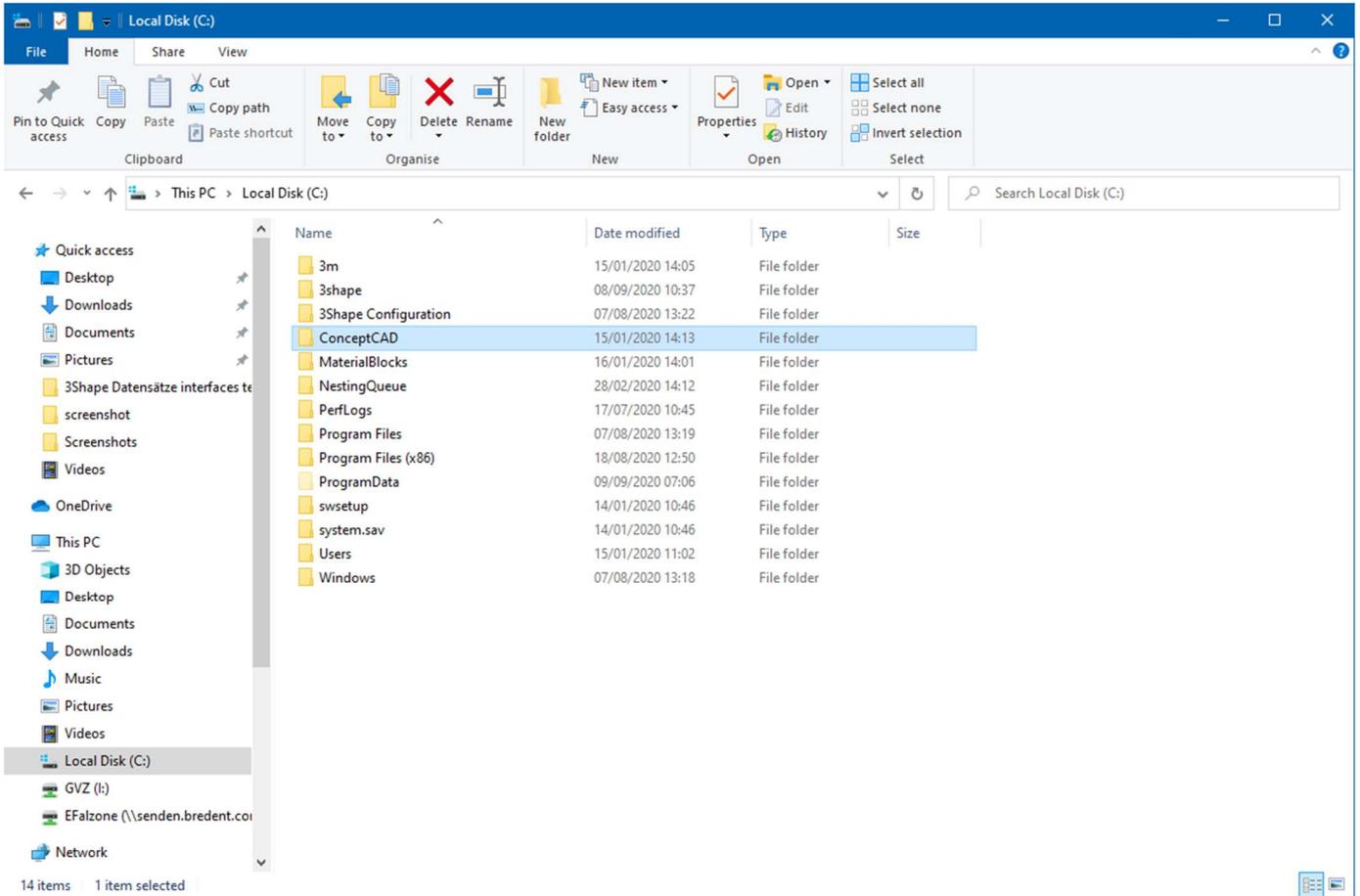

## 7: open the folder "library"

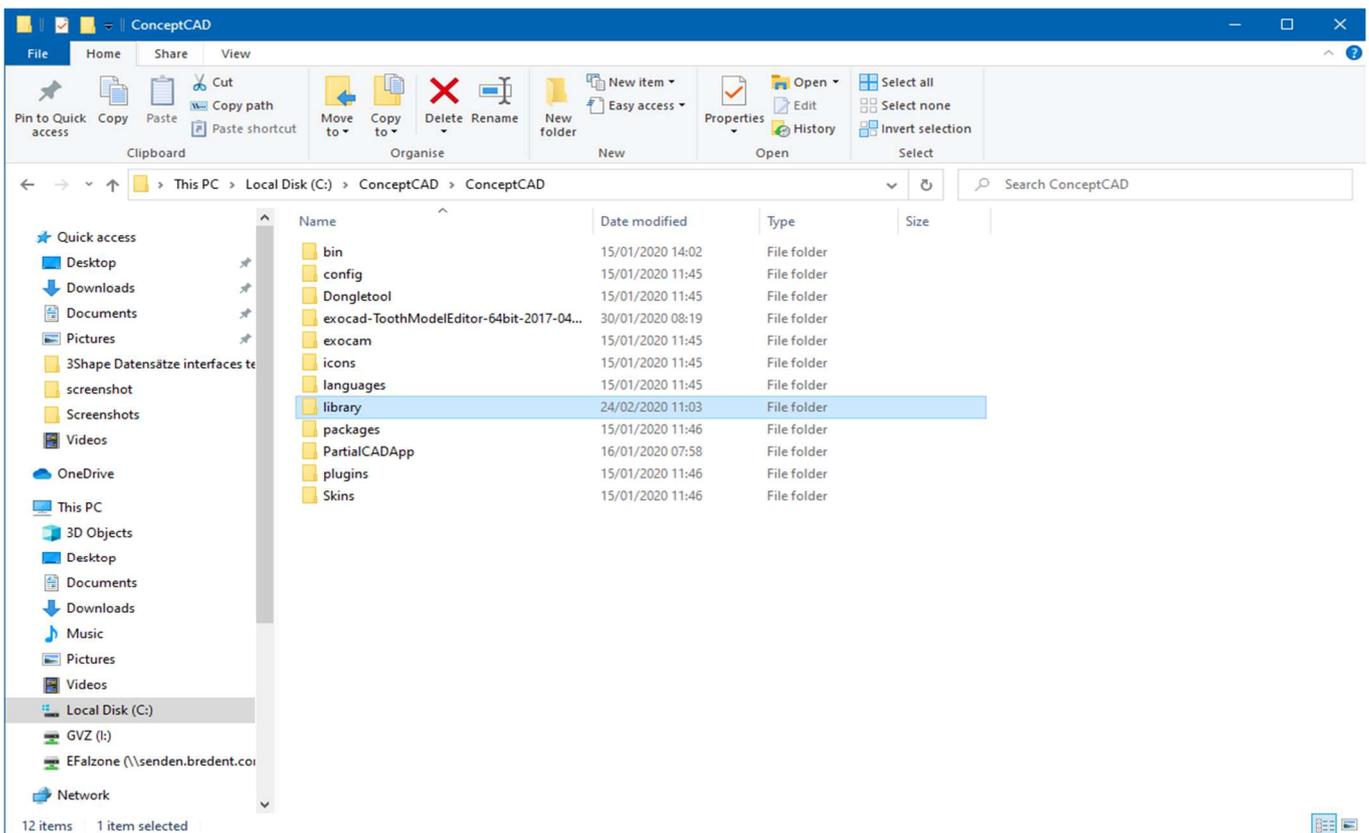

8: For implant libraries paste the files into the folder “implant”.

For model creator libraries paste the files into the folder “modelcreator-implants”.

For attachments paste the files into the folder “attachments”.

For bars paste the files into the folder “bar”.

If you have previous libraries do not forget to delete them before.

