


SKY pro guide – Übersicht Bohreranwendung

Implantat	Pilotbohrer	Twistdrill	Finalbohrer		Crestalbohrer	Implantateindreher	
			Harter Knochen D1	Mittelharter / weicher Knochen D2, D3, D4		SKY	copaSKY
5.2 mm 	PCDXLXD450	TDXL52D225	1. FD1L52D350	1. FD2L52D350	CRDXLXD400	n.a.	COPMLXD475
			2. FD1L52D400	2. FD2L52D400			
8 mm 	PCDXLXD450	TDXL52D225	1. FD1L52D350	1. FD2L52D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
			2. TDXL08D225	2. FD2L08D350			
10 mm 	PCDXLXD450	TDXL52D225	1. FD1L52D350	1. FD2L52D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
			2. TDXL10D225	2. FD2L10D350			

Fallbeispiel: * bSKY4514

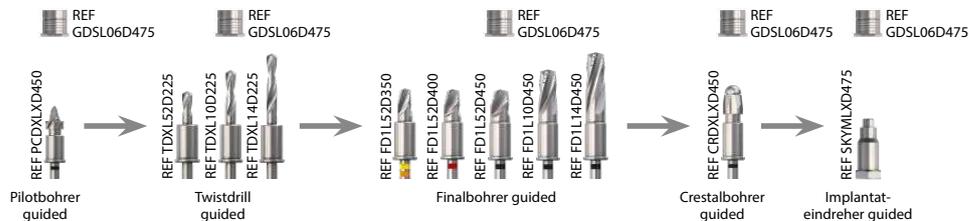

SKY pro guide – Übersicht Bohreranwendung


Implantat	Pilotbohrer	Twistdrill	Finalbohrer		Crestalbohrer	Implantateindreher					
			Harter Knochen D1	Mittelharter / weicher Knochen D2, D3, D4		SKY	copaSKY				
12 mm 	PCDXLXD450	TDXL52D225	1. FD1L52D350	1. FD2L52D350	CRDXLXD35n	SKYMLXD475	COPMLXD475				
			2. TDXL12D225	2. FD2L12D350							
			1. TDXL52D225	1. FD1L52D350				1. FD2L52D350	CRDXLXD350	SKYMLXD475	n.a.
			2. TDXL12D225	2. FD1L12D350				2. FD2L12D350	CRDXLXD400	SKYMLXD475	COPMLXD475
14 mm 	PCDXLXD450	TDXL52D225	1. FD1L52D350	1. FD2L52D350	CRDXLXD35n	SKYMLXD475	COPMLXD475				
			2. TDXL10D225	2. FD2L10D350							
			5. TDXL14D225	5. FD2L14D350							
			1. TDXL52D225	1. FD1L52D350				1. FD2L52D350	CRDXLXD350	SKYMLXD475	n.a.
16 mm 	PCDXLXD450	TDXL52D225	1. FD1L52D350	1. FD2L52D350	CRDXLXD35n	SKYMLXD475	n.a.				
			2. TDXL10D225	2. FD2L10D350							
			3. TDXL16D225	3. FD2L16D350							
			1. TDXL52D225	1. FD1L52D350				1. FD2L52D350	CRDXLXD350	SKYMLXD475	n.a.

Irrtum und Änderungen vorbehalten 0099830D-20231201






Fallbeispiel: copa4014

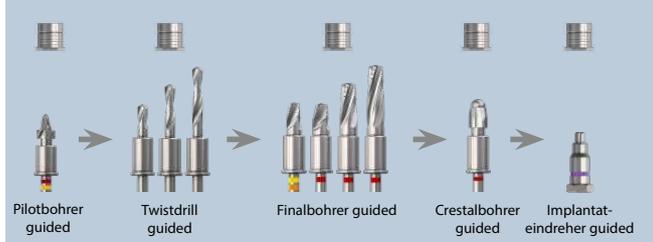

REF TPGXLXD400

- 3.5 N
- 3.5
- 4.0
- 4.5

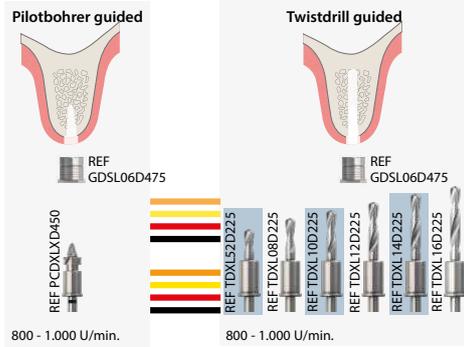

Implantatlängen

- 8, 10, 12, 14, 16 mm
- 3.5 N 3.5

- 5,2, 8, 10, 12, 14, 16 mm
- 4.0

- 5,2, 8, 10, 12, 14, mm
- 4.5

