

Exocad libraries installation procedure



ATTENTION

For the usage of the new library, it is necessary to use the NEW SCAN ABUTMENTS for SKY and copaSKY

copaSKY
REF: **COPASC12**



SKY
REF: **SKYUSC11**




If you are using the scanbodies COPAUSCI, SKYUSCAE, or SKYUSCAI, you must choose the “xxx_legacy”-libraries. Keep in mind that the new parts are not usable with the legacy libraries!

Download of the libraries

1. Visit the breident-implants website: <https://breident-group.com/breident-group-cad-library/>
2. Read the information and click on: Next




3. Enter your E-Mail-Adress, mark the agreement, give the right answer of the calculation, and click on: Subscribe and continue to CAD Library
If you want to get informed about library updates, please also mark the updates

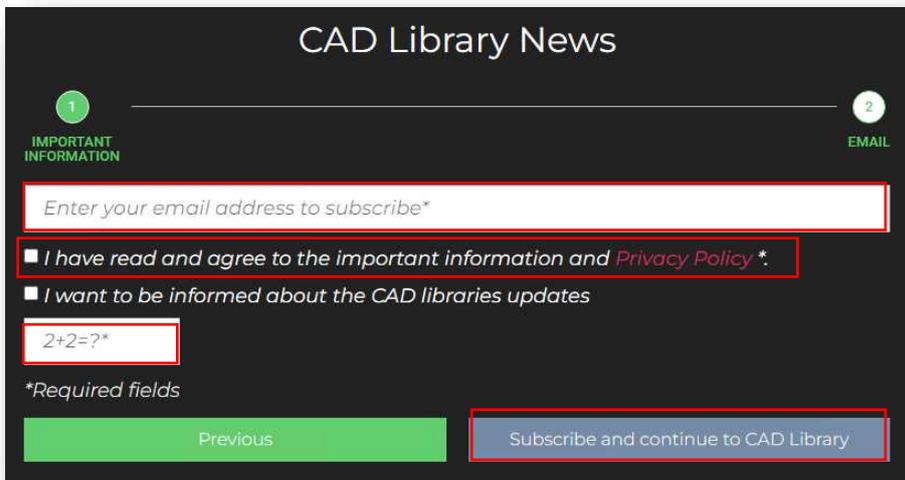

The screenshot shows a subscription form titled "CAD Library News". It is divided into two sections: "1 IMPORTANT INFORMATION" and "2 EMAIL". In the "IMPORTANT INFORMATION" section, there is a text input field for an email address, two checkboxes for agreeing to terms and receiving updates, and a simple math question "2+2=?". In the "EMAIL" section, there is a "Subscribe and continue to CAD Library" button. A "Previous" button is also visible.

4. You will be forwarded to this site and choose exocad (usually you will be on this site automatically).

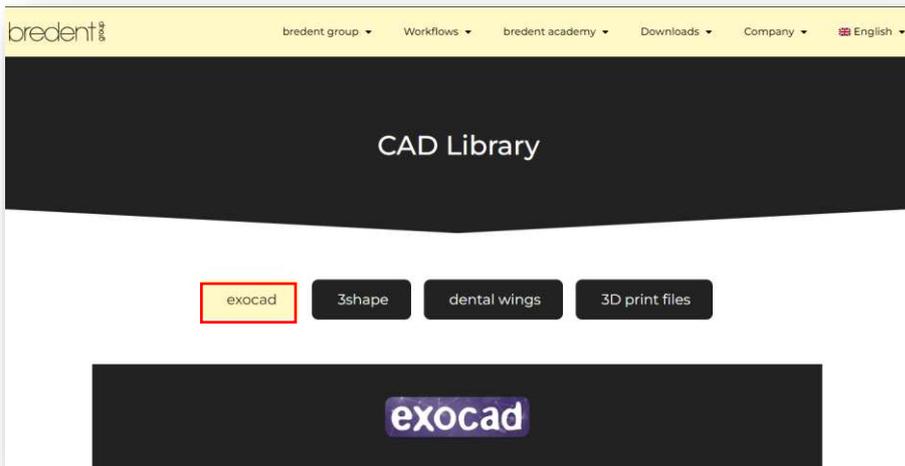

5. Choose the files to download. The download will start immediately. The easiest way will be to download the complete CAD library – so you will get all data for the work with SKY.

exocad

BioHPP SKYO elegance
bredent BioHPP elegance Prefab.zip
(File size: 110.5 MB / 08-09-2022)
BioHPP elegance Prefab Library

copa SKYO
bredent medical copaSKYO+ModelCreator libraries.zip
(File size: 53.6 MB & 1.2 MB / 23-11-2023)
DentalCAD Library
ModelCreator Library

SKYO
bredent medical SKYO+ ModelCreator libraries.zip
(File size: 90.8 MB & 1.7MB / 23-11-2023)
DentalCAD Library
ModelCreator Library

mini² SKYO
bredent medical mini²SKYO.zip
(File size: 1.6 MB & 0.55 MB / 23-11-2023)
DentalCAD Library
ModelCreator Library

white SKYO
bredent medical whiteSKYO+ ModelCreator libraries.zip
(File size: 1 MB & 0.42 MB / 23-11-2023)
DentalCAD Library
ModelCreator Library

bredent barprofile
bredent Barprofile Library For Exocad 9 18.zip
(File size: 6 KB)
bredent Barprofile Library

bredent attachment
bredent Attachment Library For Exocad 9 18
(File size: 6 MB)
bredent Attachment Library

RevoCone
bredent Revo-Cone Pose.zip
(File size: 150 KB / 08-03-2023)
bredent RevoCone Library

bredent Complete CAD library
Complete CAD library for exocad.zip
(File size: 267.3 MB / 23-11-2023)
exoCAD Library


.PDF
Exocad Libraries Installation Procedure
(File size: 1 MB)
Installation procedure

6. You will find the files in your Download folder or in the folder you have chosen.


Installation of the libraries

1. Delete all bredent files in the library folders
2. Open the zip-file and copy the marked folders into the marked folders in the exocad library path

Content of the downloaded Zip Archive

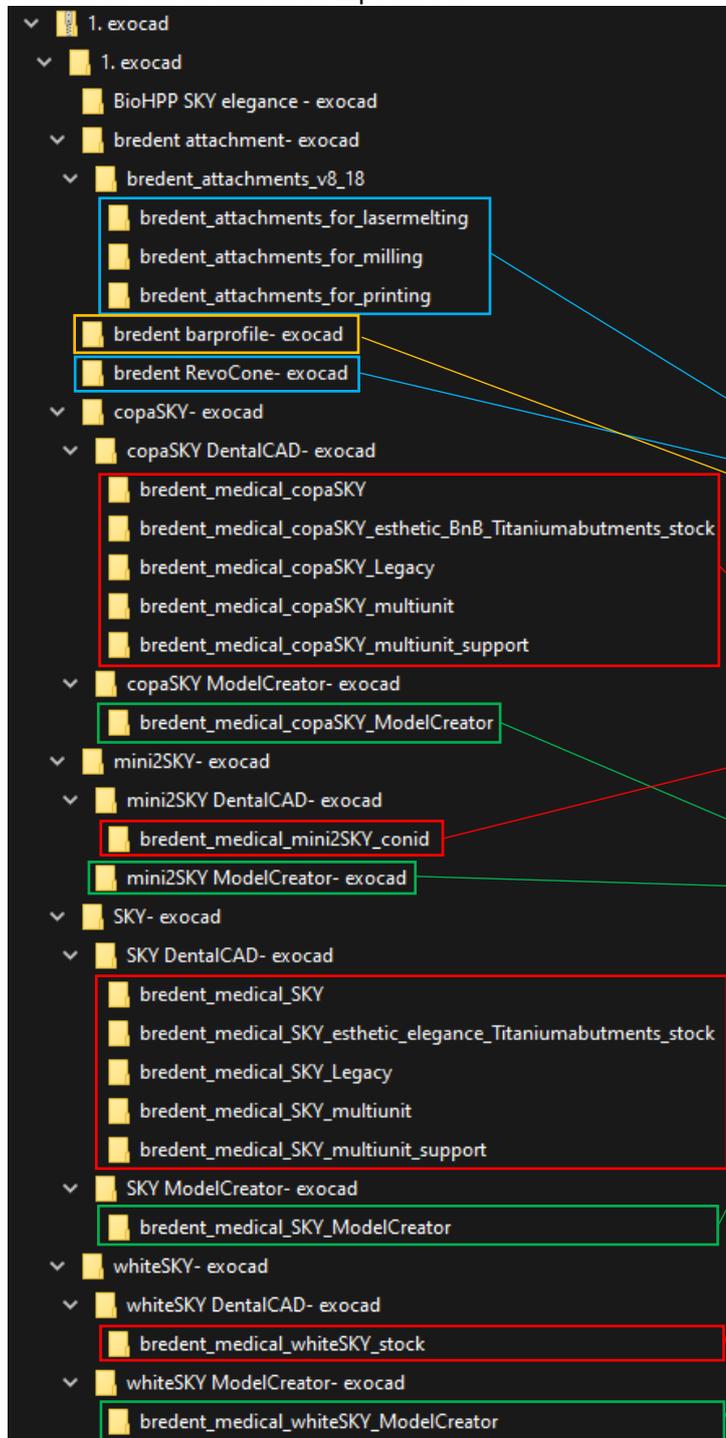

<your path to your exocad-
folder>\DentalCADApp\library\

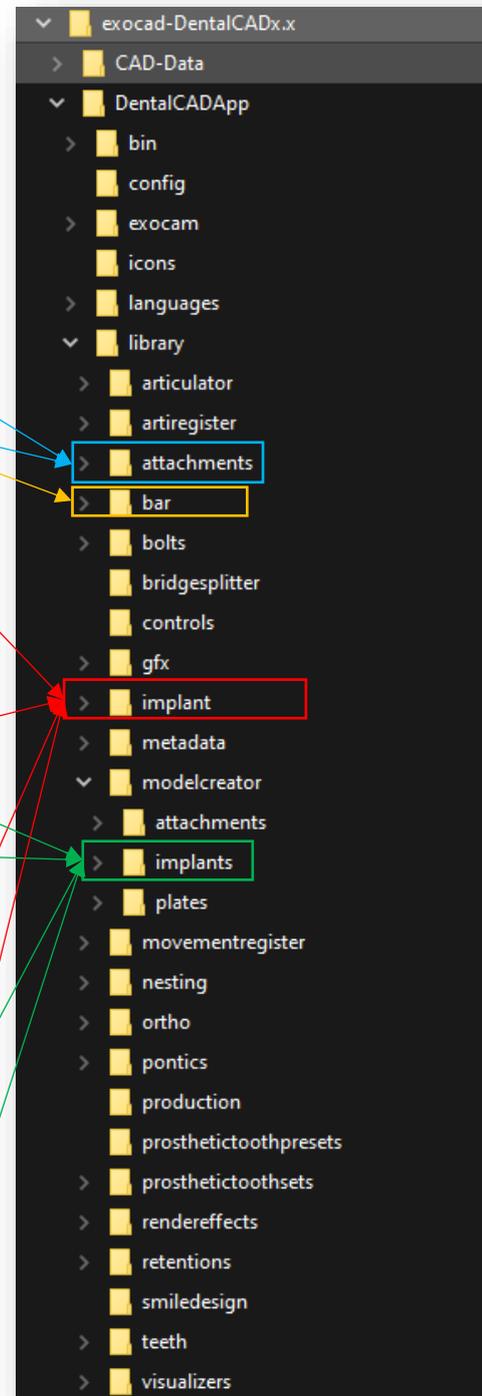

3. After restart of exocad you will find all the new libraries in exocad.