


SKY pro guide – overview of drill application

Implant		Pilot drill	Twist drill	Final drill		Crestal drill	Implant mounter	
Length	Ø			Hard bone D1	Medium hard / soft bone D2, D3, D4		SKY	copaSKY
5.2 mm 	4.0	PCDXLXD450	TDXL52D225	1. FD1L52D350 2. FD1L52D400	1. FD2L52D350 2. FD2L52D400	CRDXLXD400	n/a	COPMLXD475
	4.5		TDXL52D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450	1. FD2L52D350 2. FD2L52D400 3. FD2L52D450	CRDXLXD450	n/a	COPMLXD475
8 mm 	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL08D225	1. FD1L52D350 2. FD1L08D350	1. FD2L52D350 2. FD2L08D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	4.0		1. TDXL52D225 2. TDXL08D225	1. FD1L52D350 2. FD1L52D400 3. FD1L08D400	1. FD2L52D350 2. FD2L52D400 3. FD2L08D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5		1. TDXL52D225 2. TDXL08D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L08D450	1. FD2L52D350 2. FD2L52D400 3. FD2L52D450 4. FD2L08D450	CRDXLXD450	SKYMLXD475	COPMLXD475
10 mm 	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L10D350	1. FD2L52D350 2. FD2L10D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	3.5		1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L10D350	1. FD2L52D350 2. FD2L10D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L52D400 3. FD1L10D400	1. FD2L52D350 2. FD2L52D400 3. FD2L10D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5		1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L10D450	1. FD2L52D350 2. FD2L52D400 3. FD2L52D450 4. FD2L10D450	CRDXLXD450	SKYMLXD475	COPMLXD475

Case study: * bSKY4514

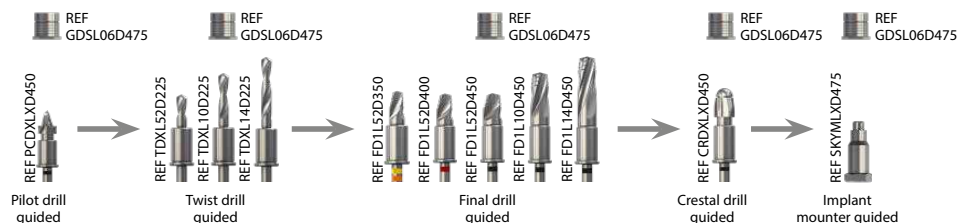

SKY pro guide – overview of drill application


Implant		Pilot drill	Twist drill	Final drill		Crestal drill	Implant mounter	
Length	Ø			Hard bone D1	Medium hard / soft bone D2, D3, D4		SKY	copaSKY
12 mm 	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L12D350	1. FD2L52D350 2. FD2L12D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	3.5		1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L12D350	1. FD2L52D350 2. FD2L12D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L52D400 3. FD1L12D400	1. FD2L52D350 2. FD2L52D400 3. FD2L12D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5		1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L12D450	1. FD2L52D350 2. FD2L52D400 3. FD2L52D450 4. FD2L12D450	CRDXLXD450	SKYMLXD475	COPMLXD475
14 mm 	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL10D225 5. TDXL14D225	1. FD1L52D350 2. FD1L10D350 3. FD1L14D350	1. FD2L52D350 2. FD2L10D350 3. FD2L14D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	3.5		1. TDXL52D225 2. TDXL10D225 5. TDXL14D225	1. FD1L52D350 2. FD1L10D350 3. FD1L14D350	1. FD2L52D350 2. FD2L10D350 3. FD2L14D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL10D225 3. TDXL14D225	1. FD1L52D350 2. FD1L52D400 3. FD1L10D400 4. FD1L14D400	1. FD2L52D350 2. FD2L52D400 3. FD2L10D400 4. FD2L14D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5 *		1. TDXL52D225 2. TDXL10D225 3. TDXL14D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L10D450 5. FD1L14D450	1. FD2L52D350 2. FD2L52D400 3. FD2L52D450 4. FD2L10D450 5. FD2L14D450	CRDXLXD450	SKYMLXD475	COPMLXD475
16 mm 	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL10D225 3. TDXL16D225	1. FD1L52D350 2. FD1L10D350 3. FD1L16D350	1. FD2L52D350 2. FD2L10D350 3. FD2L16D350	CRDXLXD35n	SKYMLXD475	n/a
	3.5		1. TDXL52D225 2. TDXL10D225 3. TDXL16D225	1. FD1L52D350 2. FD1L10D350 3. FD1L16D350	1. FD2L52D350 2. FD2L10D350 3. FD2L16D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL10D225 3. TDXL16D225	1. FD1L52D350 2. FD1L52D400 3. FD1L10D400 4. FD1L16D400	1. FD2L52D350 2. FD2L52D400 3. FD2L10D400 4. FD2L16D400	CRDXLXD400	SKYMLXD475	n/a



SKY
PRO GUIDE





Case study: copa4014

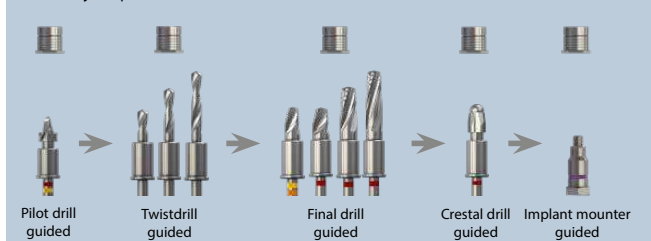

3.5N
3.5
4.0
4.5

Pilot drill guided

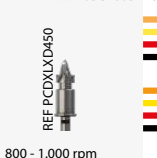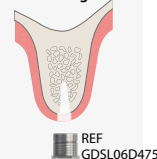

800 - 1.000 rpm

Twist drill guided



800 - 1.000 rpm

Implant length

8, 10, 12, 14, 16 mm

3.5N 3.5


5, 2, 8, 10, 12, 14, 16 mm

4.0

5, 2, 8, 10, 12, 14, mm

4.5

Implant mounter guided

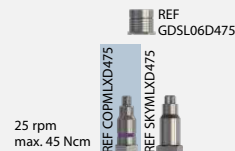

25 rpm
max. 45 Ncm

Crestal drill guided



300 rpm

Hard bone D1

Final drill guided



300 rpm

Final drill guided




3.5N 3.5

Final drill guided



4.0

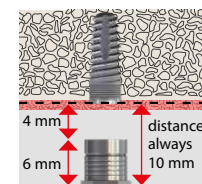

distance always 10 mm

guide sleeve




guide shaft

REF GDSL06D475

REF GDSL04D475


Medium hard / soft bone D2, D3, D4

Final drill guided

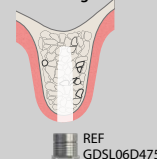


300 rpm

Final drill guided

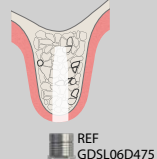

3.5N 3.5

Final drill guided

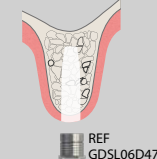

4.0

REF GDSL06D475

REF GDSL06D475

50 years of brendent

brendent
medical

brendent
group