



create natural beauty
Red-White Aesthetics

crea.lign
Veneering composite

Layering scheme for RED-WHITE individualization

Layering instructions - crea.lign GUM

1



The bone is represented with beige.

2



The effect of depth is achieved with lila.
Lila is spread from the mucolabial fold towards the teeth.

3



The layers that were previously applied are coated with pink.

4



The alveolar sockets are accentuated with rose.

5



Well perfused areas are accentuated with red.

Light is applied to the gingival margin.

6



The entire surface is sealed with transpa.

powered by
visio.lign

bredent
group



create natural beauty
Red-White Aesthetics

crea.lign
Veneering composite

Layering scheme for RED-WHITE individualization

Layering instructions - crea.lign Paste GUM

1



Use Paste PO to contour the marginal area.

2



The alveolar sockets are built up with Paste PL.

3



Use light to apply the gingival margin.

4



The effect of depth is achieved with lila.

Well perfused areas are accentuated with red.

5



The entire surface is sealed with transpa.



009485GB-20160411
Mistake and subject to change reserved

bre dent

bre dent